

# IntelViewer for MACs

## Installation and Login Guide

### Installation


#### 1. Select **Download IntelViewer for MAC**

IntelViewer will require installation on your device.  
Please click on the appropriate option below for installation.

- > Download IntelViewer for PC
- > Download IntelViewer for MAC



#### 2. Once downloaded open the installer. **Drag IntelViewer into the applications folder.**

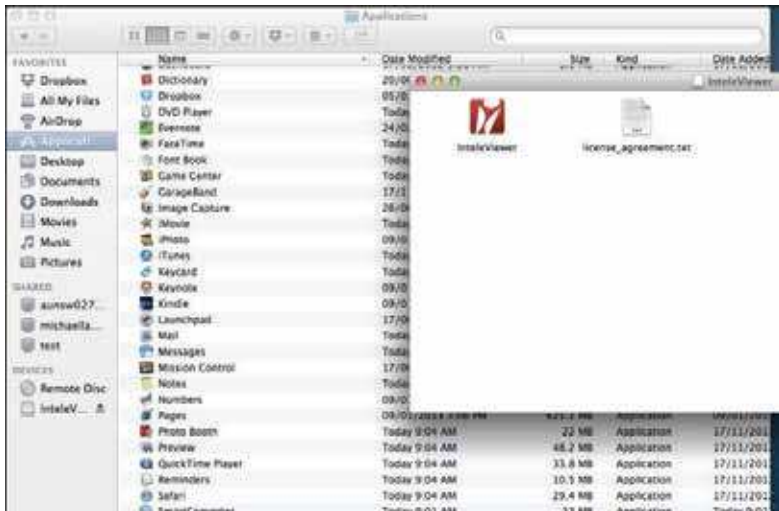


## Installation continued


- The first time you enter the application you will be prompted to Enter a New Server.  
Click **Enter a New Server**



- Type in the server URL  
<https://pacs.qld.i-med.com.au>  
Click **OK**

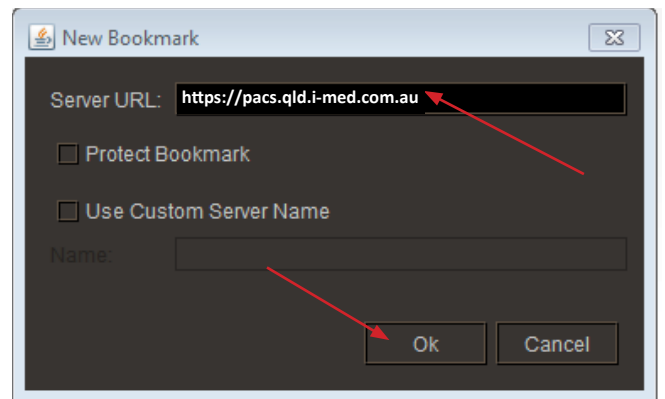

- Enter your username and password  
Click **ENTER**

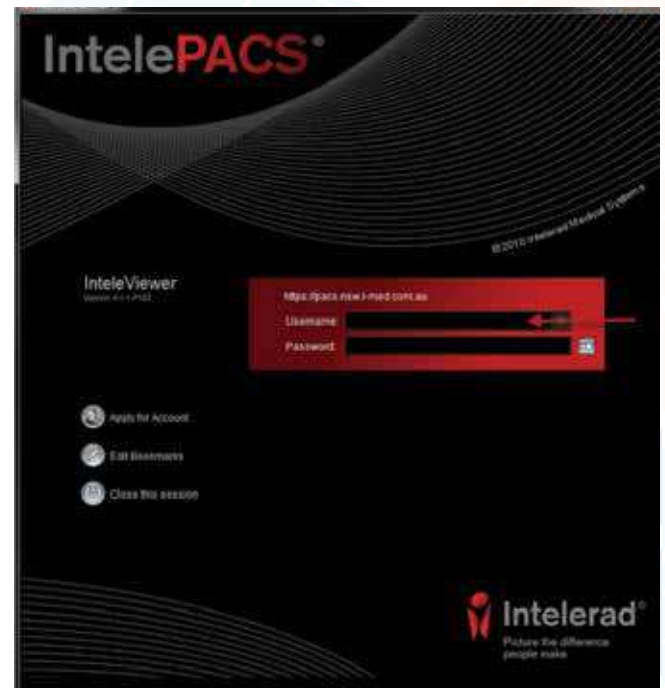

### TIPS:

Requires JAVA. If JAVA is not installed you will be prompted to install.

Apple's default security setting does not allow you to install applications from unverified sources. You can change this by going to: System Preferences/ Security & Privacy